

**Our Riboflavin solutions initialize the formation of new collagen cross-links and protect the eye against the UV-A light applied during the corneal cross-linking procedure.**



## **Peschke® TE**

### **Our new transepithelial Solution for epi-on procedure**

- No removal of the corneal epithelium necessary
- Significant reduction of pain and danger of postoperative infections
- Recommended instillation time: 20 minutes (1 drop every 2 minutes = 10 drops)
- Pre-loaded glass syringe containing 2.0 ml liquid
- Ingredients: 0.25 % Riboflavin (Vitamin B2), 1.0 % HPMC, 0.007 % Benzalkoniumchloride

## **Peschke® M**

### **Standard Riboflavin Solution without Dextran for epi-off procedure**

- Does not reduce corneal thickness
- Recommended instillation time: 20 minutes (1 drop every 2 minutes = 10 drops)
- Pre-loaded glass syringe containing 3.0 ml liquid
- Ingredients: 0.1 % Riboflavin (Vitamin B2), 1.0 % HPMC

## **Peschke® D**

### **Standard Riboflavin Solution with Dextran for epi-off procedure**

- The Dresden Original: Time proven Riboflavin solution with dextran
- Recommended instillation time: 20 minutes (1 drop every 2 minutes = 10 drops)
- Pre-loaded glass syringe containing 3.0 ml liquid
- Ingredients: 0.1 % Riboflavin (Vitamin B2), 20 % dextran 500

## **Peschke® H**

### **Hypotonic Riboflavin Solution for corneal swelling**

- To swell thin corneas (< 400  $\mu$ ) by means of osmotic effect
- Recommended instillation time: 1 drop every 5 seconds until corneal thickness has reached 400  $\mu$
- Pre-loaded glass syringe containing 1.5 ml liquid
- Ingredients: 0.1 % Riboflavin (Vitamin B2)

## **Peschke® L**

### **Riboflavin Solution for use with LASIK procedures**

- For use in connection with LASIK procedures on thin corneas
- Recommended usage: after flap preparation and excimer treatment put 4–5 drops on stroma, put flap back, wait 1–2 minutes, open flap and rinse off Riboflavin, put flap back and radiate with 30 mW for 2 minutes.
- Pre-loaded glass syringe containing 1.95 ml liquid
- Ingredients:  $\geq$  0.23 % Riboflavin (Vitamin B2)



**PESCHKE Trade GmbH**

Boesch 73 · 6331 Hünenberg · Switzerland  
Phone +41 41 784 9460 · Fax +41 41 784 9462  
info@peschketrade.com · www.peschketrade.com

**PESCHKE TRADE**  
MedTech is our passion